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#### Nephrophan – dialyser membrane

#### a new development of the VEB Filmfabrik Wolfen Fotochemisches Kombinat

The quick increase of the effectivity of the dialysis aggregates is forced by reason of the constantly raising demand for dialysis capacity for treatment of acute and chronic insufficiency of the kidneys. The efficiency of the dialysers is determined in decisive manner by the quality of the used membranes.

As result of practical knowledge for many years, the dialysis membrane

#### Nephrophan

was developed by the Filmfabrik Wolfen Fotochemisches Kombinat, being prominently suitable for all artificial kidneys in the figure of a spool and of a folding, and working adequate to the flexible pipe principle.

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Schönfelder –

Schönfelder –

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**Nephrophan – dialyser membrane**  
**a product of high quality**

Nephrophan is characterized by three advantages:

- a specific structure being conditioned by the manufacturing process,
- the extraordinary small wall thickness of the cellulose membranes,
- the important flexibility and thereby negligible risk of perforation.

By reason of the ingenious co-operation of these three characteristics, excellent dialysis and ultrafiltration results are achieved with Nephrophan at sufficient strength.

**Example no. 1:**

**Half-life values of different substances<sup>1)</sup>**

| Test solution      | t/2 (min) |
|--------------------|-----------|
| 300 mg % urea      | 109       |
| 20 mg % uric acid  | 192       |
| 20 mg % creatinine | 174       |
| 80 mg % glucose    | 354       |

**Example no. 2:**

At a length of the flexible pipe of 4 metres of Nephrophan and at an active surface of the flexible pipe of about 0.4 m<sup>2</sup> on occasion of a clinical test, ultrafiltration rates between 500 and 700 ml/h could be obtained adequate to the blood pressure and the hydration condition of the patient. By reason of producing a low pressure within the system of the rinsing solution of the dialysis, the values has been increased to above 1,000 ml/h.<sup>2)</sup>

**Nephrophan – dialyser membrane**  
**carefully examined and comprehensively clinically tested**

By reason of toxicological and pharmacological expertises, arguments against a putting into practice at patients are not raised. The overall view is completed through microbiological and haematological special investigations.

Before starting the production, Nephrophan was clinically examined in detail about for one year by means of a large-scale test within all kidney centres of the German Democratic Republic. Thereby extensive experience was gathered

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at many thousands of patient-dialyses in connection with the dialyser "Aue II".<sup>3)</sup> With a dialysis time of 10 hours, a sufficient treatment effect was achieved at patients with acute and chronic failure of the kidneys.

**Nephrophan – dialyser membrane**  
**always the same quality by reason of continuous supervision**

A permanently unchanged quality is ensured owing to a continuous production supervision within up-to-date furnished laboratories. An additional guarantee for permanent dialysis and ultrafiltration results are afforded in virtue of continuously performed medical supervisions.

Nephrophan – dialyser membrane

**Manufacturer:** VEB Filmfabrik Wolfen  
Fotochemisches Kombinat  
444 Wolfen 1  
German Democratic Republic

**References:**

- <sup>1)</sup> H. Klinkmann, M. Holtz, W. Willgerodt, G. Wilke, D. Schoenfelder – An improved variation of the cellulose membrane. V. Conference E. D. T. A., Dublin 1968.
- <sup>2)</sup> H. Klinkmann, M. Holtz, B. Osten, W. Willgerodt, G. Wilke, D. Schoenfelder – Clinical practical knowledge with the artificial kidney "Aue II". Till now unpublished.
- <sup>3)</sup> W. Willgerodt, H. Klinkmann, M. Holtz, W. Richter, G. Wilke, D. Schoenfelder – Medizintechnik **8** (1968) A 4, 131–136.

**Technical data**

Nephrophan is an endless cellulose flexible pipe, which may be at present short-termly delivered within the following dimensions:

- Unit of package: Rollers with 200 metres in each case
- Flat width: 65 mm and 45 mm
- Wall thickness: on the average about 14 . . . 15  $\mu$

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